

EP/CPR300S Advanced Full Body CPR Training Manikin W/ Printer



Implement standard: 2015 guideline for CPR.

Features:

1. Simulate standard open airway and sound prompting.
2. External breast compression: indicator light display, digital counter display and sound prompting.
 - a. Indicator light display of correct and wrong compression position; digital counter display; sound prompting of wrong compression.
 - b. Display of correct (at least 5 cm) and wrong (less than 5 cm) compression intensity; digital strip indicator light (yellow, green, red) shows compression depth; counter display; sound prompting of wrong operation.
3. Artificial respiration (Inhalation) indicator light display, digital counter display and sound prompting:
 - a. Inhalation is $\leq 500\text{ml}/600\text{ml}-1000\text{ml}$, strip indicator light shows inhalation volume; counter display of correct and wrong operations, and sound prompting of wrong operation.
 - b. Inhalation too quickly or too much result in air entering into stomach, counter display and sound prompting of wrong operation.
4. Ratio of compression and artificial respiration: 30:2 (one or two person).
5. Operating cycle: one cycle includes five times of 30:2 ratio of compression and artificial respiration.
6. Operation frequency: at least 100 times per minute.
7. Operation methods: exercise operation; examine operation.
8. Operation time: counting down device.
9. Print: print operation result.
10. Examination of pupil response: mydriasis and myosis.
11. Examination of carotid response: simulate spontaneous carotid pulse in compression process.
12. Working conditions: Input power is 110-240V.

Main Components:

- | | |
|-----------------------------|---|
| 1. Full body manikin (1) | 5. CPR face shield sheet (50pcs/box) |
| 2. Monitor (1) | 6. Exchangeable lung bag (5) |
| 3. Portable plastic box (1) | 7. Exchangeable face skin (1) |
| 4. CPR operation pad (1) | 8. Temperature sensing print paper (2 roll) |
| | 9. Guide manual (1) |

